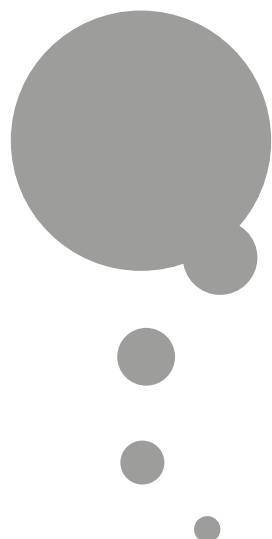
<p>Modelo: Saline REF Troquel: TRO-PROS-014 Dimensiones: 540 x 175 mm</p>	<p>Colores:</p> <p>black</p> 
<p>Código: PROS583</p>	

PROS583

AVIZOR

saline

175 mm

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**COMPOSICION:**

Polihexanida 0,00005%, en solución isotónica, tamponada y estéril.

**gb** RINSING AND STORING SOLUTION FOR ALL TYPE OF CONTACT LENSES. PREVENTS FORMATION OF CALCIUM DEPOSITS. ALSO FOR DISSOLVING ENZYMIC TABLETS

**INSTRUCTIONS FOR USE:**

- Before handling your lenses, wash your hands.
  - After the cleaning process, and if necessary, place the lens on the palm of your hand and rinse thoroughly with AVIZOR SALINE SOLUTION.
  - To dissolve the enzymatic tablet use the solution according to instructions for use of tablets.
- Last revision date: 2009/11

**PRECAUTIONS:**

- Once opened, the bottle should not be used after 60 days.
- Maintain the bottle well closed after use.
- Do not use the product if the safety seal is missing or broken.
- You may store your lenses up to 30 days in the closed lenscase
- Do not ingest the solution and keep away from children's reach.
- Keep at room temperature.
- Do not allow the dropper to come into contact with any surface to avoid contamination
- In case of irritation, consult your contact lens specialist.
- Before using any ophthalmic medicine, inform your doctor you are a contact lens user.
- Do not use if you are allergic to any of the components.
- Never use tap water to rinse your lenses or lens case.
- Do not use after the expiration date.

**COMPOSITION:**

Polyhexanide 0,00005 % in a buffered, isotonic and sterile solution.

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