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VISAID CleanSoft are single-use eye wipes with hyaluronic acid, Centaurea and Centella Asiatica extracts.

VISAID CleanSoft are sterile, single-use coated hygienic wipes for the palpebral fissure (waterline), eyelids and eyelashes, and contain hyaluronic acid, Centaurea and Centella Asiatica extracts.

COMPOSITION:

Aqua, Propanediol, Glycerin, Sodium Hyaluronate, Centaurea Cyanus Flower Extract, Centella Asiatica Extract, Disodium Phosphate, Sodium Phosphate, PEG-40 Glyceril Cocoate, Sodium Coceth Sulfate, Caprylyl Glycol, Sodium Chloride, Methylpropanediol, Phenylpropanol, Disodium EDTA, Potassium Sorbate.

Tested under dermatological and ophthalmological control.

The wipes contain:

Hyaluronic acid (in the form of sodium hyaluronate): a substance that lubricates, moisturises and protects the skin.

Centella Asiatica extract: has soothing and regenerative properties when applied to the skin.

Centaurea extract: soothing, decongesting, protective and toning effects.

Mild surfactant: facilitates the removal of rheum, dirt and excess sebum.

Among the excipients are non-irritating ingredients with emollient and hydrating properties.

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INDICATIONS:

VISAID CleanSoft is intended for:

Gentle cleansing of the eye area by gently removing any substances adhered to the eyelids and eyelashes. Suitable for adults, children and babies.

INSTRUCTIONS FOR USE:

Wash your hands. Use one wipe for each eye, as many times a day as necessary (at least morning and evening). Rinsing is unnecessary.

Eyelid and eyelash hygiene:

Open the sachet and remove the coated wipe. With the eye closed, apply to the eyelid and eyelashes, massaging gently in a circular motion, to loosen and remove any residue.

Palpebral fissure (waterline) hygiene:

Facing a mirror, hold the lid open by pressing on the eyelashes with a finger. Roll up the wipe and pass lightly over the edge of the eyelid several times, taking care not to brush the surface of the eye.

PRESENTATION:

20 packs of single-use wipes

Manufactured by Laboratorios Entema S.L., Passatge Llobregat Nau 8. 08184, Palau-solità i Plegamans, Barcelona for **Avizor S.A., La Cañada, 13-17. 28850, Torrejón de Ardoz, Madrid**